

# DISINFECTANT *Solutions*



## Quick DEFENSE™ AQ DISINFECTANT

ONE STEP CLEAN AND DISINFECT



- Disinfects a variety of viruses and bacteria in as little as 60 seconds
- Convenient and ready-to-use spray formula
- Effectively clean and disinfect hard, non-porous surfaces
- Fragrance Free
- On EPA List N

**S**late®  
Since 1911

CARE FOR WORK ENVIRONMENTS®

# Quick DEFENSE™ AQ DISINFECTANT

## FAST-ACTING SOLUTION

Quick Defense™ AQ Disinfectant is an all-in-one solution when you need rapid disinfection and cleaning capabilities in one product. This ready-to-use disinfectant has a variety of kill claims and has proven to be an effective virucide, bactericide, tuberculocide and fungicide, even in the presence of soils. The alcohol and quaternary-based active ingredients are working at the same time to deliver fast-acting results. Offering a 60-second dwell time on most claims, this disinfectant reduces time spent cleaning while increasing facility health standards. This conveniently packaged disinfectant will make keeping your facility germ-free a breeze.

Keep employees and visitors safe and your facility clean by frequently disinfecting high-touch surfaces. This broad-spectrum disinfectant is sure to keep your facility healthy, clean and disease-free.

## MARKETS

Being a hospital-grade disinfectant, Quick Defense AQ Disinfectant is the perfect disinfectant solution for a variety of healthcare markets. The 60-second contact time offers fast disinfection on high touch surfaces throughout the facility. This makes turning rooms around a breeze. This is the perfect disinfectant for use in hospitals and critical care areas.

Utilize Quick Defense AQ Disinfectant throughout your hospital, including ambulances, CAT labs, emergency rooms, labor and delivery, recovery rooms and more.

Critical care units require fast and effective disinfection. Quick Defense AQ Disinfectant is the perfect solution for bed railings, carts, chairs, counters, IV poles, IV pumps, hospital beds, traction devices, MRI, CAT, scales, paddles and workstations on wheels (WOW). This product can be used to clean and decontaminate external surfaces of ultrasound transducers and probes, prior to any required high level disinfection or sterilization.

## KILL CLAIMS

Quick Defense AQ Disinfectant has 48 kill claims with a 60-second contact time. This product is effective against viruses, bacteria and fungi such as:

- Rotavirus Strain WA
- Canine parvovirus
- HIV-1
- Herpes Simplex Virus Type 1
- MRSA
- VRSA
- E-Coli
- Athlete's Foot
- ...and more

Quick Defense AQ Disinfectant (EPA Registration #88494-3-70799) has demonstrated effectiveness against viruses similar to SARS-CoV-2 on hard, non-porous surfaces. Therefore, Quick Defense AQ Disinfectant can be used against SARS-CoV-2 when used in accordance with the directions for use against Poliovirus Type 1; Rhinovirus virus on hard, non-porous surfaces. Please refer to the CDC at <https://www.cdc.gov/> for more information.

## COMMON APPLICATIONS

Quick Defense AQ Disinfectant is the perfect solution when you need a fast turnaround and can be applied to a variety of hard, non-porous surfaces including:

- Bed railings
- Cabinets
- Desks
- Gym equipment
- Chairs
- Countertops
- Doorknobs
- Bathroom Fixtures

FOR PLASTIC AND PAINTED SURFACES, SPOT TEST ON AN INCONSPICUOUS AREA BEFORE USE. DO NOT USE ON ACRYLIC PLASTICS, COATED OPTICAL SURFACES, UNPAINTED WOOD OR LEATHER.

## DIRECTIONS FOR USE

1. Always wear personal protective equipment, including gloves and safety glasses.
2. Pre-clean desired surface to be treated by removing all visible soils and debris.
3. Hold bottle upright, about 6 to 8 inches away from surface.
4. Spray area until thoroughly saturated.
5. Allow treated surface to remain wet for 60 seconds.
6. Let air dry.

## TECHNICAL DATA:

**REGISTRATIONS:** EPA Registration Number 88494-3-70799

**APPEARANCE:** clear and colorless

**ODOR:** alcohol

**DISINFECTANT TYPE:** ethyl alcohol and quaternary ammonium compound

**pH:** 3.7

**FLASHPOINT:** 65.3°F

Please review label and SDS on [stateindustrial.com](http://stateindustrial.com) for all product directions, precautions and first aid information.

## PACKAGING

128601

QT CS12



**State Industrial Products**  
5915 Landerbrook Drive  
Mayfield Heights, OH 44124  
To Order Call: 1-866-747-2229  
[www.stateindustrial.com](http://www.stateindustrial.com)

**Puerto Rico**  
Royal Industrial Park, Bldg "M"  
Local #5, Carr 869, km 1.5 Palmas  
Cataño, P.R. 00962  
To Order Call: 787-275-3185

